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The screenshot shows the 'Program parameters' dialog box with the 'Recon' tab selected. The dialog is organized into several sections:

- Cardiology protocols:** Includes fields for 'Study 1', 'Study 2', and 'Study 3'. It has radio buttons for 'Primary non-gated', 'Primary gated', and 'Secondary non-gated'. A 'Secondary NG' radio button is selected. A 'Count limit [%]' dropdown is set to '20'. There are also radio buttons for 'Outline' and 'Extend transverse FOV'.
- Lung protocols:** Includes fields for 'Study 1' and 'Study 2'.
- Neurology protocols:** Includes fields for 'Primary', 'Secondary', and 'Tertiary' (set to 'hermes\_rec\_default'). Radio buttons for 'Perform secondary reconstruction' and 'Perform tertiary reconstruction' are present.
- Oncology protocols:** Includes fields for 'Primary' (set to 'onco\_rec\_default'), 'Secondary' (set to 'onco\_nac\_rec\_default'), and 'Tertiary' (set to 'hermes\_rec\_default'). Radio buttons for 'Perform secondary reconstruction', 'Perform tertiary reconstruction', and 'Reconstruct whole FOV in NAC' are present.
- Mumap display zoom [%]:** Set to '100'. 'Uniform mumap display zoom [%]:' is set to '265'.
- Warning options:** Radio buttons for 'Skip subset warning', 'Skip energy window warning', 'Skip frame of reference warning' (selected and highlighted with a red box), and 'Skip mumap truncation warning'. There is also an option to 'Reduce number of reconstruction methods'.
- Mumap transformation:** Radio buttons for 'Automatic full 6 parameter', 'Automatic translation only' (selected), and 'Manual'.
- Mumap page screen cap label:** An empty text field.

A 'Save' button is located at the bottom of the dialog.

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Hermes Medical Solutions Ltd
Hermes Medical Solutions, Inc
Hermes Medical Solutions Canada, Inc
Hermes Medical Solutions Germany GmbH