

SYSTEM ENVIRONMENT REQUIREMENTS

Document number: PC-007 System Environment Requirements Rev.8_EN
Document revision date: 10/10/2023



Table of content

1	GENERAL INFORMATION	3
1.1	WARNING	3
2	VERIFIED SYSTEM COMBINATIONS	4
2.1	APPLICATION SERVERS	4
2.2	APPLICATION SERVER CLIENTS	4
3	CONTACT INFORMATION.....	5

1 GENERAL INFORMATION

Hermes Medical Solutions' software have been verified on systems with the specifications described in this document. For questions regarding other combinations, please contact the support service, see *Contact information*.

For our reconstruction software there are specific requirements which are described in separate columns and supersedes the general requirements.

Hermes Medical Solutions welcomes feedback from readers of this manual, please report any errors in content or typography and suggestions for improvements to our support, see *Contact information*.

NOTE: It is the customer's responsibility to install and maintain anti-virus software on the server and client computers and apply the necessary protection against potential threats.

1.1 Warning



Only properly trained service personnel of Hermes Medical Solutions, or an authorized dealer, shall perform installations, configuration, and service of this product.



No other, than Hermes Medical Solutions approved, applications shall be installed on the computer device for which Hermes Medical Solutions applications are intended to be used. Use of other applications may result in impaired performance and, in the worst case, incorrect output data.

2 VERIFIED SYSTEM COMBINATIONS

For all our products - Hermes licensing software: ExplorerPWD V4.5.2 or later.
All graphic cards (GPU) mentioned below are from NVIDIA®.

2.1 Application Servers

Operating System	Graphics Card
Microsoft® Server 2016	Quadro RTX™ x000

CPU: Intel®

RAM: Minimum 64GB

Citrix VDA: Citrix Virtual Apps 7 1912 LTSR (at least CU2)
Citrix Virtual Apps 7 2203 LTSR (at least CU2) (recommended)

2.2 Application Server Clients

Monitor Resolution: Maximum tested resolution: 1920x1200@60Hz

Remote Access: Citrix Workspace app 1912 LTSR (at least CU7)
Citrix Workspace app 2203.1 LTSR (at least CU2) (recommended)
No other remote access software should be installed

Network speed: Minimum 100 Mbps (recommended 1Gbps)

3 CONTACT INFORMATION

Contact any of the addresses below for service, support or if you have any other questions:

**Hermes Medical Solutions AB**

Strandbergsgatan 16
112 51 Stockholm
SWEDEN
Tel: +46 (0) 819 03 25
www.hermesmedical.com

General e-mail address:

info@hermesmedical.com

Support e-mail addresses:

support@hermesmedical.com
support.ca@hermesmedical.com
support.us@hermesmedical.com

Authorized representatives**UK Responsible Person**

Hermes Medical Solutions Ltd
Cardinal House
46 St. Nicholas Street
Ipswich, IP1 1TT
England, United Kingdom

CH Authorized Representative

CMI-experts
Grellinger Str. 40
4052 Basel
Switzerland

Sales and service offices**Hermes Medical Solutions Ltd**

York Suite, 7-8 Henrietta Street
Covent Garden
London WC2E 8PS
UK
Tel: +44 (0) 20 7839 2513

Hermes Medical Solutions, Inc

710 Cromwell Drive, Suite A
Greenville, NC27858
USA
Tel: +1 (866) 437-6372
Fax: +1 (252) 355-4381

Hermes Medical Solutions Canada, Inc

1155, René-Lévesque O., Suite 2500
Montréal (QC) H3B 2K4
Canada
Tel: +1 (877) 666-5675
Fax: +1 (514) 288-1430

Hermes Medical Solutions Germany GmbH

Robertstraße 4
48282 Emsdetten
Deutschland
Tel: +46 (0)819 03 25