HERMIA

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Product name: Hybrid Viewer

Release version: 7.2.0
 Marketing name: Hermia
 Software build no: 150

CE 2862

Rx only
 2025-07-01
 UDI (01)00859873006240(8012)007002000

MD Medical device
 eIFU Indicator
<https://www.hermesmedical.com/ifu>
 support@hermesmedical.com
 Canada: support.ca@hermesmedical.com
 USA: support.us@hermesmedical.com

HERMES MEDICAL SOLUTIONS
 Hermes Medical Solutions AB
 Strandbergsgatan 16
 112 51 Stockholm
 SWEDEN

CH REP CMI-experts, Grellinger Str. 40,
 4052 Basel, Switzerland

Renderer:
 NVIDIA RTX A2000 8GB Laptop GPU/PCIe/SSE2

Copyright:
 NetBSD
 Copyright (c) 2001-2007 The NetBSD Foundation, Inc.
 All rights reserved.
 This code is derived from software contributed to The NetBSD Foundation by Simon Burge, Luke Meeburn, and Christos Zoulas.
 Redistribution and use in source and binary forms, with or without modification, are permitted provided that the following conditions are met:
 1. Redistributions of source code must retain the above copyright notice, this list of conditions and the following disclaimer.
 2. Redistributions in binary form must reproduce the above copyright notice, this list of conditions and the following disclaimer in the documentation and/or other materials provided with the distribution.
 3. All advertising materials mentioning features or use of this software must display the following acknowledgement:
 This product includes software developed by the NetBSD Foundation, Inc. and its contributors.
 4. Neither the name of The NetBSD Foundation nor the names of its contributors may be used to endorse or promote products derived from this software without specific prior written permission.
 THIS SOFTWARE IS PROVIDED BY THE NETBSD FOUNDATION, INC. AND CONTRIBUTORS "AS IS" AND ANY EXPRESS OR IMPLIED WARRANTIES, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE ARE DISCLAIMED. IN NO EVENT SHALL THE FOUNDATION OR CONTRIBUTORS BE LIABLE FOR ANY DIRECT, INDIRECT, INCIDENTAL, SPECIAL, EXEMPLARY, OR CONSEQUENTIAL DAMAGES (INCLUDING, BUT NOT LIMITED TO, PROCUREMENT OF

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Modification of the product is not allowed and may result in hazardous situations.

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Only properly trained service personnel by an authorized dealer or by Hermes Medical Solutions, shall perform installations, and service of this product.

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No other, than Hermes Medical Solutions approved, applications shall be installed on the computer device for which Hermes Medical Solutions applications are intended to be used. Use of other applications may result in impaired performance and, in the worst case, incorrect output data.

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Security

If the network is unavailable it may not be possible to maintain the Intended Use of the device.

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Generic

Changing patient or study information can lead to changes in quantitative results

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Generic

The intended user should not rely solely on the output identified by Hybrid Viewer NM Processing™ but should perform a full systematic review and interpretation of the entire patient dataset.

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Generic

User provided protocols, scripts and programs are not validated nor warranted by Hermes Medical Solutions. The party using such programs is solely responsible for the results

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Generic

If the CT has a larger slice thickness than the PET, some PET slices are skipped when scrolling on the fused images or when using Quick ROI. The workaround is to either set the PET as the master volume, or scroll in the PET only view.

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Generic

Due to the fact that there is no Radiopharmaceutical vector in the Dicom header, as there is for EnergyWindow, we cannot guarantee which pharmaceutical applies to which image for multi-isotope studies. Therefore the Acquisition Info option for overlay of Radiopharmaceutical for Planar studies should not be used for multi-isotope studies

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**Generic for all NM Processing and BRASS**

The quantitative results generated should be used only as an aid to diagnosis and under no circumstances should they be used without interpreting and analyzing the original images.

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Generic for all NM Processing

To minimize the risk of errors in processing, it is recommended to always use one of the predefined protocols distributed with the Hermia software, or one configured on site with Hermes Medical Solutions.

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Generic for all NM Processing

If an NM Processing application is used to follow up a patient, and earlier studies for this patient have been processed in an older version or another application, these studies should be reanalyzed in the current version of the application before any comparison between previous and current studies is performed. Comparison of studies processed with different applications or versions, can result in inaccurate quantification and may lead to misdiagnosis.

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Generic for all NM Processing

The intended user should not rely solely on the output identified by Hybrid Viewer NM Processing™, but should perform a full systematic review and interpretation of the entire patient dataset.

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First Pass

It is important to note whether the SVC curve represents a good bolus i.e. is a short curve with a sharp peak, as this will affect the accuracy of the results. If the curve has several peaks, the application should not be used. If it has a prolonged peak (> 3seconds wide at 10% of the peak2), it may be possible to use SVC deconvolution).

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**Renogram**

Patlak Analysis should not be used if the acquisition was started late.

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**Renogram**

A good quality bolus and resultant input curve is imperative when using Patlak methodology.

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**Renogram**

If Renogram Analysis is used for following up a patient, and earlier studies for this patient have been processed in an older version or another application, these studies should be reanalyzed in the current version of Renogram Analysis before any comparison between previous and current studies is performed. Comparison of studies processed with different applications or versions can result in inaccurate quantification and may lead to misdiagnosis.

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**Renogram**

Normalize perfusion To Max: Scale renogram perfusion curve to the max value: This option should not be used when processing transplant studies with a protocol which calculates Perfusion index. The index would not be affected but the iliac and transplant curves would not be correct

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**Renogram**

Relative Function (Retention): Values for the relative function are calculated as the ratio of the amplitudes of the retention plateau during the uptake phase. It is not recommended to use this method as the results may be unreliable.

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**Renogram**

GFR Gates and ERPF Arroyo methods are not valid for transplant kidneys, as the depth correction used in the calculations is not accurate for anterior acquisitions. ERPF Bubeck can be used for transplant kidneys as this method does not require depth correction.

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BRASS

If the current version of BRASS™ is used for following up a patient, and EARL/ENC-DAT studies for this patient have been processed in an older version of BRASS™, these studies shall be reanalyzed in the current version of BRASS™ before any comparison between previous and current studies is performed. Comparison of studies processed with different versions of BRASS™ or with different databases, can result in inaccurate quantification and may lead to misdiagnosis.

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BRASS

Important to read System Environment Requirements. Only the computer devices and application specified shall be used. Use of other applications and computer devices may result in impaired performance and, in the worst case, incorrect diagnosis of patients.

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BRASS

A study which has been processed with a previous version of BRASS should be processed with the updated version, to make sure the same quantitative values are being produced. This should be done for studies of each type used in BRASS (HMPAO,FDG etc).

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Thyroid

If Thyroid Analysis is used for following up a patient, and earlier studies for this patient have been processed in an older version or another application, these studies should be reanalyzed in the current version of Thyroid before any comparison between previous and current studies is performed. Comparison of studies processed with different applications or versions can result in inaccurate quantification and may lead to misdiagnosis.

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Dosimetry

This application should not be used with versions of OLINDA/EXM earlier than version 2.1.

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Dosimetry

It is important that the scan lengths and speed be the same between time points to avoid quantification errors.

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Dosimetry

The wholebody scans must always be acquired with the same speed and length when using a planar calibration factor.

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Dosimetry

Calibration technique: First WB: In cases where this method is used and no quantitative SPECT is available, it is important that the scan lengths and speed be the same between time points to avoid quantification errors.

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Dosimetry

If a single Background ROI is drawn, (second in the ROI list underneath the Standard ROI), this ROI will be used to subtract background from all ROIs except the Standard. Care should be taken to draw it in a place which is suitable as a generic background. The results are very sensitive and if the ROI is not correctly positioned the curve fitting will be incorrect.

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Lung V/Q

The image subtraction and ratio calculations made by this program assume that the ventilation and perfusion SPECT images are reconstructed with the same algorithm and settings. Large differences in settings between ventilation and perfusion reconstructions, such as different matrix size, different corrections applied or large differences in iterations and subsets or different post-filtering parameters might cause unintended effects in the ratio and Q-Corrected images. The Q-Corrected images and the VQ Ratio images should always be interpreted in conjunction with the original unprocessed ventilation and perfusion images.

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DICOM SEG

Volumes of regions viewed in Hybrid Viewer may not perfectly match those displayed in external DICOM viewing applications for the same region. This is due to differences in the voxel grid used to define segmentations in different applications, and quantitation methods for voxels on region boundaries. This may affect dose map region statistics which use all region voxels, for example mean dose, especially for smaller regions.

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GENERAL APPLICATION MESSAGES

CRITICAL:

- `vdconf.xml` file was not found. Application could not start.
- User protocol doesn't exist. Application could not start.
- Protocol doesn't exist. Application could not start.
- OpenGL version 2.0 or higher is required to use shaders.
- No data loaded.
- Problem exporting statistics
- Problem exporting table results to cvs file
- Export directory doesn't exist. Saving study is not possible.
- Dynamic/Gated data is not supported
- Need power of 2 color table size
- Program will not launch due to insufficient memory. Please quit some other applications and try again
- Studies or SC were not loaded successfully. The file format is not supported or the data is corrupted.

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GENERAL APPLICATION MESSAGES

WARNING:

- Print spool doesn't exist. Printing and Saving will not be possible.
- Spool directory doesn't exist. Saving Result File is not possible.
- Spool directory doesn't exist. Saving Movie is not possible.
- Prints are not saved correctly.
- Dynamic Splash Zoom value had been defined in common protocol. This value is not the same that value defined in properties. We recommend to save your properties with this value.
- There are defined movie groups that are not created and saved. Do you want to save them before closing?
- Coregistration parameters will be saved in the result file. It is not necessary to save registered studies. You have to reload original data with the application and result file.
- `xx` doesn't exist. Saving CSV statistics is not possible
- Spool directory doesn't exist. Saving Screen Capture is not possible.
- Data too large to perform filtering on this system.
- A number in the image order cannot be duplicated. The label images order will not be applied.
- The image order number cannot be larger than the number of images

loaded. The label image order will not be applied.

- All transformations applied, like masking values, rotation, flipping, duplicate an image or adding an empty image, will be lost.
- PNG files are not saved correctly.
- Missing or invalid information.
- Studies or SC were not loaded successfully. Time step is not the same, although Data will be displayed. Computed values may be affected.
- You should update your Lung Reprojection layout protocol to the latest version.

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SYNC MESSAGES

WARNING:

- Semi-automatic alignment cannot be applied to the images from the opposing view because one or more ANT and POST images do not have identical date and time.
- Data have not been registered due to different frame of reference.
- Fusion may not be appropriate.
- Fusion may not work properly!
- Frames of reference are different, fusion may not be appropriate.

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MOTION CORRECTION MESSAGES

WARNING:

- The two series don't have same acquisition date and time. Do you still want to apply motion correction to this series?

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REPORT TOOL MESSAGES

WARNING:

- The report has been modified.\n" "Do you want to save your changes?"

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ROI/VOI MESSAGES

WARNING:

- You are not drawing on the right study.
- Refresh display prior to copy rois/vois.
- Non-transverse VOIs exist for this dataset. Only transverse VOIs can be saved as RT Structure Sets.
- RTP Export", "Unable to save to RT Structure set - Not available for Interfile Data.
- Non-transverse VOIs exist for this dataset. Only transverse VOIs can be saved as DICOM SEG.
- Unable to save to DICOM SEGMENTATION - Not available for Interfile Data.

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ROI RATIO MESSAGES

CRITICAL:

- You should have one first static image
- You should have one second static image

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SAVE AVI MESSAGES**WARNING:**

- Unable to Create Compatible DC.
- SetErrorMessage(_T("Unable to Create Heap")).
- Unable to Allocate Memory on Heap.
- Unable to Create the Movie File.
- Unable to Create Video Stream in the Movie File.
- Unable to Create Compressed Stream: Check your CODEC options.
- Unable to Set Video Stream Format.
- Unable to Write Video Stream to the output Movie File.

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- Lung VQ calculations not performed as Perfusion and Ventilation series cannot be identified. Check string matching in user protocol.
- Lung Ratio can not be performed. Ventilation and Perfusion series are not selected correctly in the layout protocol.
- The total counts of the ventilation study are greater than the total counts of the perfusion study so this application is probably unsuitable for these studies.
- No energy information in %1, perfusion correction will be applied.

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BRASS**CRITICAL:**

- No valid license for template.
- Database missing for template.
- Database signature invalid for template.
- BRASS will not work with a dynamic PET.
- At least one SPECT or PET study must be loaded.
- No Template Loaded or Missing Dataset for Difference.
- Missing Dataset for Difference.

WARNING:

- Anatomy patient will not be registered/displayed.
- BrassRegistration folder is not present. Registration can't be applied.
- Problem exporting brass region map counts.

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- No "POST" image found. Cannot run.

WARNING:

- Large difference in Left/Right kidney uptake or kidneys too close. Please check kidney and background positions.
- If only one kidney drag the missing kidney cross out of the image.
- Same string matching has been detected for ANT and POST Studies. The protocol will be saved anyway.
- ANT Labels and POST Labels must be defined for string matching.
- No Static Dataset Found with String Matching.

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DMSA**CRITICAL:**

- You can't load different modalities (Planars/Dynamics/Volumes) at the same time.
- ANT Labels and POST Labels must be defined for string matching.

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- No Static Dataset Found with String Matching.
- No Dynamic Dataset Found with String Matching.
- No POST image found.
- No Volume Dataset Found with String Matching.
- No strings matched, loading default view.

WARNING:

- ANT Study and POST Study were detected as the same Study using string matching. Only POST study will be used.
- It is not possible to remove all frames.
- Motion Correction can not be applied to Ant Study as Post and Ant series don't have same acquisition date and time.
- Dynamic acquisition loaded, you need to enable saving summed dynamic in properties.
- Same string matching has been detected for ANT and POST Studies. The protocol will be saved anyway

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DOSIMETRY**CRITICAL:**

- Error reading isotopes or phantoms files.
- The Dicom fields ActualFrameDuration (0018,1242) and/or NumberOfFramesInRotation (0054,0053), required for SPECT calibration, are missing in at least one SPECT study. Application will exit.
- Unable to locate Olinda.exe.
- An Olinda version prior to V2.1 has been detected and should not be used with this application. Update Olinda to V2.1 or later.
- No Whole Body dosimetry patient found with string matching defined in protocol. And no SPECT/Quantitative SPECT were loaded.

WARNING:

- Residence time bigger than whole body residence time. It could lead to incorrect dose calculation.
- Be sure the calibration factor is set correctly and the studies correctly aligned.
- Effective half-life is greater than physical half-life, so physical half-life will be used.
- Image data from only one time point was detected. Physical decay of the

isotope will be assumed unless external dose rate measurements are entered.

- ANT/POST mismatch in string matching.
- Calculated values may be incorrect if the series date/time has been changed.
- External dose-rate option has been selected and more than one study has been loaded.
- Triple Energy Window Scatter Correction will be applied.
- Photopeak only will be used.
- ROIs have not been copied. Copy them using 'Copy Rois/Vois' in the ROI/VOI window.
- Probe values have not been set.
- A pre-determined residence time for Red Marrow has been entered in the user properties. Red marrow ROIs/VOIs will be ignored.
- Some lesions have been drawn, but no masses are associated with those lesions. Enter masses in 'Target Organ Masses'.
- As no vois are drawn, SPECT will not be used.
- Moved/Edited/Rotated local ROIs will be reset to their original position/form

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GALBLADDER**CRITICAL:**

- Loaded patients do not have same image size, they can not be combined.
First patient will be displayed
- No Dynamic Dataset Found with String Matching
- No Static Dataset Found with String Matching
- No Static/Dynamic Dataset Found with String Matching
- It is not possible to load 2 series from the same time point together
- Two static studies required for processing.
- A dynamic study label is assigned to a static study
- A static study label is assigned to a dynamic study

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GASTRIC EMPTYING**CRITICAL:**

- A dynamic study label is assigned to a static study
- A static study label is assigned to a dynamic study
- ANT Labels and POST Labels must be defined for string matching
- No Static Dataset Found with String Matching
- Conflict in String Matching. It has not been possible to differentiate POST and ANT
- No Dynamic Dataset Found with String Matching
- Number of Static Images Mismatch for ANT/POST
- Number of Dynamic Acquisitions Mismatch for ANT/POST

WARNING:

- Calculated values may be incorrect if the series date/time has been changed.
- Motion Correction can not be applied to POST Study as ANT and POST series don't have same acquisition date and time
- Same string matching has been detected for ANT and POST Studies. The protocol will be saved anyway

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LUNG QUANTIFICATION

CRITICAL:

- ANT Labels and POST Labels must be defined for string matching
- No Static Dataset Found with String Matching

WARNING:

- All transformations applied, like masking values, rotation, flipping, duplicate an image or adding an empty image, will be lost.
- Same string matching has been detected for ANT and POST Studies. The protocol will be saved anyway

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PARATHYROID

CRITICAL:

- Application will not run if parathyroid string matching is not defined
- Application will not run if thyroid string matching is not defined
- No parathyroid patient found with string matching
- No thyroid patient found with string matching

WARNING:

- Dynamic study loaded. Enable 'Save summed dynamic study' in Results section user protocol.
- Motion Correction can not be applied to Parathyroid Study as Thyroid and Parathyroid series don't have same acquisition date and time
- Motion Correction can not be applied to Thyroid Study as Thyroid and Parathyroid series don't have same acquisition date and time
- Same string matching has been detected for parathyroid and thyroid studies. The protocol will be saved anyway.

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REMNANT LIVER

CRITICAL:

- ANT Labels and POST Labels must be defined for string matching
- No Dynamic Dataset Found with String Matching

WARNING:

- No Post Dynamic Dataset Found with String Matching
- No Ant Dynamic Dataset Found with String Matching
- Same string matching has been detected for ANT and POST Studies. The protocol will be saved anyway

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RENOGRAM

CRITICAL:

- No Geometric Mean study was detected using string matching. Renogram will be applied on Post series
- It is possible to load 2 series from the same time point together only if the option to create Geometric Mean is selected or if the 2 series are Geometric Mean and Post.
- It is not possible to load 2 series from the same study at the same time unless the Geometric Mean option is selected.
- An incorrect label has been matched against a study.
- You should have geometric mean study for flow/renogram/orthostase and diuretic.
- No Dynamic Dataset Found with String Matching, application could not start
- Post/Ant patients have not been detected correctly using string matching for %1. No Geometric Mean patients can be computed
- No Post patients were detected using string matching. %1 cannot run with only Ant Views
- No Post/Ant patients were detected using string matching for %1. No Geometric Mean patients can be computed

WARNING:

- Motion correction seems to be applied and study not saved. If you want to reload result file with saved motion correction study, you need to save the study before. Click Ignore to continue and to save result file.
- Kidney depth is missing in Patient Information Widget.
- Injected Activity information is missing in Patient Information Widget.
- Loaded patients do not have same image size, they can not be combined. First patient will be displayed.
- Patlak/OutFlow/Retention graphs will not be displayed on Compare Tab.
- Pharmaceutical found in study header is not the same as the one defined in properties. Study header will be used.
- Result will be saved in more than one xml file.
- All calculations will be made from Geometric Mean study. Depth Correction and GFR/ERPF will be disabled.
- Calculated values may be incorrect if the series date/time has been changed.
- A maximum frame time of 20 seconds (throughout the study) is required

to calculate Patlak/Outflow and Retention.

- An acquisition of at least %1 seconds with a maximum frame time of 20 seconds (throughout the study) is required to calculate Patlak/Outflow.
- An acquisition of at least %1 seconds with a maximum frame time of 20 seconds (throughout the study) is required to calculate Retention.
- A maximum frame time of 20 seconds (throughout the study) is required to calculate Retention. Lasix part will not be used.
- An acquisition of at least 20 minutes with a maximum frame time of 20 seconds (throughout the study) is required to calculate Retention.
- All studies don't have same pixel size. Combined study can't be computed.
- Please select a new range, intercept value must be > 0. Background subtraction and values related to patlak will not be computed.
- There are prints in the Print Window that are not saved/printed.\n" "Do you want to save/print them?

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SALIVARY**CRITICAL:**

- No Static Dataset Found with String Matching.

WARNING:

- Same string matching has been detected for Pre/Post Lemon studies. The protocol will be saved anyway.

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SeHCAT**CRITICAL:**

- You should have two ANT standard images
- You should have two POST standard images
- You should have four standard images
- You should have two ant or post images
- You should have two ANT Background images
- You should have two POST Background images
- You should have four Background images
- You should have two ant or post images
- You should have four Patient images
- No strings are defined for the study
- Duplicate strings are chosen. Please select again
- No strings are selected. Would you like to continue?
- You should have two ant or post images

WARNING:

- If you want to save the new string matching, you need to open the properties and set the new string matching/detector numbers

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THYROID

CRITICAL:

- Application will not run if thyroid string matching is not defined
- No thyroid patient found with string matching
- Date/Time is missing from one study. Application could not start and will exit

WARNING:

- Reference image not detected from string matching. Camera efficiency value stored in user protocol will be used instead
- Full and empty images not detected from string matching. Syringe activity measured from external counter
- Isotope was not detected in study header. Please select the correct isotope from the drop down menu
- The Reference activity dose was not detected in the study. Please enter the value manually
- The Thyroid activity dose was not detected in the study. Please enter the value manually
- Full and empty images not detected from string matching. Syringe activity measured from external counter

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ESOPHAGEAL TRANSIT REFLUX

WARNING:

- Same string matching has been detected for transit and reflux studies. The protocol will be saved anyway

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QUALITY CONTROL

CRITICAL:

- Problem exporting uniformity history.
- No Acquisition Dataset Found with String Matching.
- Phantoms have not been detected correctly.
- Phantom has not been detected correctly.

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BONE 3 PHASE

CRITICAL:

- No Dynamic Dataset Found with String Matching, application could not start
- No Static Dataset Found with String Matching
- A dynamic study label is assigned to a static study
- A static study label is assigned to a dynamic study

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COLONIC TRANSIT

CRITICAL:

- ANT Labels and POST Labels must be defined for string matching.
- Number of Images Mismatch for ANT/POST.
- You have loaded only one study, you don't have enough data.

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FIRST PASS

CRITICAL:

- No Dynamic Dataset Found with String Matching.

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FUGA

CRITICAL:

- No Planar Gated Dataset Found with String Matching.

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HIDA

CRITICAL:

- No Dynamic Dataset Found with String Matching.
- No Static Dataset Found with String Matching.

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LUNG LIVER SHUNT

CRITICAL:

- Lung Liver Ratio calculations are disabled as Lung and Liver series cannot be identified. Suggest setting the 'Use string matcher'.

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SI JOINT

CRITICAL:

- No Static Dataset Found with String Matching

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