



# RELEASE NOTES

**OLINDA/EXM<sup>®</sup>**

Version 2.4.0

**Document Name:** P58-085 Release Notes OLINDA EXM 2.4.0 Rev.2\_EN

**Document revision date:** 2025-04-30

These Release Notes inform users of news and improvements in OLINDA EXM 2.4.0, as well as any known issues to be aware of.

This is an electronic document, a copy of which can be downloaded from [www.hermesmedical.com/ifu](http://www.hermesmedical.com/ifu). Hard copies of Instructions for Use, System Environment Requirements, and Release Notes are available for free (as many as number of purchased licenses) upon request.

The Release Notes and the medical device software itself is copyrighted and all rights are reserved by Hermes Medical Solutions. Neither the software nor the manual may be copied or in any other way reproduced without prior consent in writing from Hermes Medical Solutions who reserve the right to make changes and improvements to the software and the manual at any time.

Hermes Medical Solutions\*, HERMIA\*, the HERMIA logotype\* and SUV SPECT\* are trademarks of Hermes Medical Solutions AB.

Third-party trademarks as used herein are the property of their respective owners, which are not affiliated with Hermes Medical Solutions.

\*Subject to registration in some markets

Table of content

1 INTRODUCTION ..... 3

1.1 ASSOCIATED DOCUMENTATION .....3

1.2 COMPLAINTS AND SERIOUS INCIDENTS.....3

2 NEWS AND IMPROVEMENTS ..... 4

2.1 NEW FEATURES IMPLEMENTED IN OLINDA/EXM 2.4.0.....4

2.2 PROBLEMS FIXED AND MINOR ENHANCEMENTS IN VERSION 2.4.0.....4

3 KNOWN ISSUES ..... 5

4 CONTACT INFORMATION..... 6

4.1 MANUFACTURER CONTACT INFORMATION.....6

4.2 REPRESENTATIVES .....6

4.3 SUBSIDIARIES.....6

# 1 INTRODUCTION

This document contains the public Release Notes for OLINDA/EXM® 2.4. It provides a list of new features, enhancements and bug fixes since last release.

The document also includes known issues. Every user should become familiarized with these known issues. Contact the manufacturer for any questions about the content.

## 1.1 Associated documentation

- P58-078 Instructions For Use OLINDA 2.4.0 Rev.2

The Instructions For Use contains the necessary basic information to configure the application with your own preferences.

A user guidance, intended to assist users in using the software, is available from the Help function in the software itself.

Warning messages are now listed in both the Instructions For Use and the user guidance. The warning messages clearly describe intended users, limitations in the software and the risks of making changes to the software.

## 1.2 Complaints and serious incidents

Any serious incident that has occurred in relation to the device must be reported to our support, see Contact Information.

Depending on applicable regulations, serious incidents may also need to be reported to national authorities. For the European Union, serious incidents must be reported to the competent authority of the European Union Member State in which the user and/or patient is established.

Hermes Medical Solutions welcomes feedback from users of the software and readers of this document. Please report any errors and suggestions for improvements in the software, and in the content or typography of this document, see Contact Information.

## **2 NEWS AND IMPROVEMENTS**

### **2.1 New Features implemented in OLINDA/EXM 2.4.0**

OpenApps license support has been added.

### **2.2 Problems fixed and minor enhancements in version 2.4.0**

No problems were fixed, and no enhancements were made in this version.

### 3 KNOWN ISSUES

There are no known issues related to patient safety in this version of OLINDA/EXM®.

Below is a selection of open known issues relevant to the end users.

All issues have been risk analyzed and have been classified as acceptable.

- Olinda sphere model with decay series only shows last nuclide in report (case# 64702)
- GI organs are unexpectedly set to zero (case# 62536)
- Branching Ratios not respected in decay series (case# 55257)
- OLINDA (standalone) will not license (Node Lock) if you install via custom install path during installation (case# 45898).
- Dose report does not contain the list of isotopes in a decay series when using OLINDA sphere model (case# 39146).
- Organ list displayed is not the same one as the one output in the results when modifying data (case# 44215).
- The kinetic table value for urinary bladder is not updated correctly when using voiding bladder model.
- Modify Data not working as expected when using the Apply button.

## 4 CONTACT INFORMATION

Contact any of the addresses below for service, support or if you have any other questions.

### 4.1 Manufacturer contact information



**Head office**  
**Hermes Medical Solutions AB**  
Strandbergsgatan 16  
112 51 Stockholm  
SWEDEN  
Tel: +46 (0) 819 03 25  
www.hermesmedical.com

**General e-mail address:**  
info@hermesmedical.com

**Support e-mail addresses:**  
support@hermesmedical.com  
support.ca@hermesmedical.com  
support.us@hermesmedical.com

### 4.2 Representatives

#### Authorized representatives

**UK Responsible Person**  
Hermes Medical Solutions Ltd  
Cardinal House  
46 St. Nicholas Street  
Ipswich, IP1 1TT  
England, United Kingdom

#### CH Authorized Representative

CH	REP
----	-----

CMI-experts  
Grellinger Str. 40  
4052 Basel  
Switzerland

### 4.3 Subsidiaries

**Hermes Medical Solutions Ltd**  
7-8 Henrietta Street  
Covent Garden  
London WC2E 8PS, UK  
Tel: +44 (0) 20 7839 2513

**Hermes Medical Solutions, Inc**  
2120 E. Fire Tower Rd, #107-197  
Greenville, NC27858  
USA  
Tel: +1 (866) 437-6372

**Hermes Medical Solutions Canada, Inc**  
1155, René-Lévesque O., Suite 2500  
Montréal (QC) H3B 2K4  
Canada  
Tel: +1 (877) 666-5675  
Fax: +1 (514) 288-1430

**Hermes Medical Solutions Germany GmbH**  
Robertstraße 4  
48282 Emsdetten  
Deutschland  
Tel: +46 (0)819 03 25